

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department

DATE 1-30-2009 TIME 17:22

INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *W.C. Will*

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number: 80-001858
01/30/2009

17:22:13

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:22:40
Cal Check	0.101	17:22:59
Air Blank	0.000	17:23:28

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department

DATE 1-30-2009 TIME 17:26

INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE *W.C. Will*

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

01/30/2009

17:26:26

Standard Lot#: 808401G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:27:17
Diagnostic Test	Pass	17:27:44
Air Blank	0.000	17:28:12
0.100 Cal Check	0.100	17:28:31
Air Blank	0.000	17:29:00
Subject Test	0.000	17:29:17
Air Blank	ABT*	17:29:42
Air Blank	0.000	17:30:11

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

01/30/2009

17:31:24

Standard Lot#: 808401G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:32:27
Diagnostic Test	Pass	17:32:54
Air Blank	0.000	17:33:21
0.100 Cal Check	0.101	17:33:40
Air Blank	0.000	17:34:10
Subject Test	0.000	17:34:48
Air Blank	0.000	17:35:15
Five Minute Wait	ABT*	
Air Blank	0.000	17:35:50

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
01/30/2009 17:36:30

Standard Lot#: 808401G
Last Changed By: W. WILL #974

OAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:37:19
Air Blank	0.000	17:37:47

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

01/30/2009

17:37:52

Standard Lot#: 808401G

Last Changed By: W. WILL

#974

QAS: W. WILL

GOODYEAR PD

#974

Operator: W. WILL

GOODYEAR PD

#974

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	17:38:32
Air Blank	0.000	17:39:00

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
01/30/2009 17:41:10

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:42:06
Diagnostic Test	Pass	17:42:34
Air Blank	0.000	17:43:01
0.100 Cal Check	0.101	17:43:20
Air Blank	0.000	17:43:49
Subject Test	0.076*	17:46:58
Air Blank	0.000	17:47:29
Air Blank	0.000	17:47:57
Subject Test	XXX**	17:48:29
Air Blank	0.000	17:49:04

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department

DATE 1-30-2009 TIME 17:49

INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

01/30/2009

17:49:38

Standard Lot#: 808401G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 17:50:05
Cal Check 0.100 17:50:24
Air Blank 0.000 17:50:53

Operator's Signature



Scott Specialty Gases
Air Liquide America Specialty Gases LLC

Shipped From: 6141 EASTON ROAD, BLDG 1
PLUMSTEADVILLE PA 18949-0310
Phone: 800-331-4953
PO BOX 310
Fax: 215-766-7226

C E R T I F I C A T E O F A N A L Y S I S
EBS - ETHANOL BREATH STANDARD

CMI PROJECT #: 01-78049-001
316 EAST NINTH STREET PO#: 372067
RECEIVING DEPARTMENT ITEM #: 0104EBS1000 105
OWENSBORO KY 42303 DATE: 31Mar2008

ANALYTICAL ACCURACY: +/- 2%
PRODUCT EXPIRATION: 30Mar2010

SCOTT LOT#: 808401G
ETHANOL IN NITROGEN

<u>COMPONENT</u>	<u>PPM</u>	<u>(BAC)</u>
ETHANOL	260.5	(0.100)
NITROGEN	BAL	

<u>REFERENCE STANDARD</u>	<u>CYLINDER</u>	<u>CONCENTRATION PPM</u>
NIST TRACEABLE STANDARDS *	ALM043565	262.0

* CERTIFICATION TRACEABLE TO N.I.S.T. RGM ETHANOL STANDARDS

*Project #
01-78049-001*

*808401G
EXP 30 MAR 2010*

MANUFACTURED DATE: 24Mar2008

SCOTTY SIZE: 105

APPROVED BY:

[Signature]
SARAH LAWRENCE