

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST JEFF ROGERS AGENCY GOODYEAR P.D.
DATE 7-1-06 TIME 1846
INTOXILYZER SERIAL # 80-001858 LOCATION JUSTICE FACILITY

AR 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- AR 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
AR 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
AR 4. Air blank completed.
AR 5. Calibration check completed. Test results 0. 099 AC.
AR 6. Air blank completed.
AR 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] # 1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number: 80-001858

07/01/2006

18:46:58

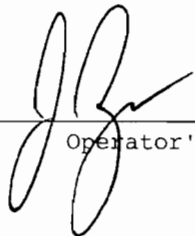
Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:47:26
Cal Check	0.099	18:47:45
Air Blank	0.000	18:48:14

 #1123

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST JEFF ROGERS AGENCY GOODYEAR P.D.
DATE 7-1-06 TIME 1850
INTOXILYZER SERIAL # 80-001858 LOCATION JUSTICE FACILITY

☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ☒ 1. Clock time check.
☒ 2. Date check.

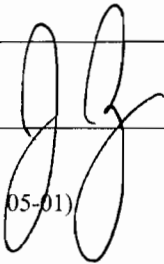
LOT 5364011

OPERATIONAL TESTS

- ☒ 1. Alcohol-free subject test result 0. 000 AC.
☒ 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
☒ 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
☒ 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE  #1123

INTOXILYZER 8000
Location: GOODYEAR JUSTICE FAC
Serial Number : 80-001858
07/01/2006 18:59:43

Standard Lot#: 536401I
Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/0101
Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:00:38
Diagnostic Test	Pass	19:01:06
Air Blank	0.000	19:01:33
0.100 Cal Check	0.099	19:01:52
Air Blank	0.000	19:02:21
Subject Test	0.054*	19:05:30
Air Blank	0.000	19:06:02
Air Blank	0.000	19:06:29
Subject Test	XXX**	19:06:53
Air Blank	0.000	19:07:27

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/01/2006

18:58:07

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Operator: J. ROGERS #1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	18:58:44
Air Blank	0.000	18:59:13

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/01/2006

18:55:58

Standard Lot#: 536401I

Last Changed By: M. SLONEKER

#6559

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	18:57:11
Air Blank	0.000	18:57:40

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/01/2006

18:50:44

Standard Lot#: 536401I

Last Changed By: M. SLONEKER

#6559

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:52:09
Diagnostic Test	Pass	18:52:36
Air Blank	0.000	18:53:03
0.100 Cal Check	0.099	18:53:22
Air Blank	0.000	18:53:52
Subject Test	0.000	18:54:13
Air Blank	0.000	18:54:41
Five Minute Wait	ABT*	
Air Blank	0.000	18:55:20

*Sequence Aborted

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST JEFF ROGERS AGENCY GOODYEAR P.D.
DATE 7-1-06 TIME 1912
INTOXILYZER SERIAL # 80-001858 LOCATION JUSTICE FACILITY

- ☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ☒ 4. Air blank completed.
- ☒ 5. Calibration check completed. Test results 0. 099 AC.
- ☒ 6. Air blank completed.
- ☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] # 1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number: 80-001858

07/01/2006

19:12:38

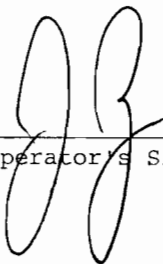
Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:13:05
Cal Check	0.099	19:13:24
Air Blank	0.000	19:13:54

 #1123

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1134
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR JUSTICE FACILITY

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0.099 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 536401I
EXP 1-3-08

SIGNATURE [Signature] # 1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number: 80-001858

07/06/2006

11:34:30

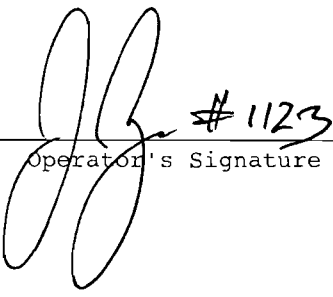
Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:34:57
Cal Check	0.099	11:35:16
Air Blank	0.000	11:35:46

 #1123

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1142
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR JUSTICE FACILITY

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
(✓) 2. Date check.

LOT 536401Z
EXP 1-3-08

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/06/2006

11:42:09

Standard Lot#: 536401I

Last Changed By: M. SLONEKER

#6559

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:43:08
Diagnostic Test	Pass	11:43:35
Air Blank	0.000	11:44:02
0.100 Cal Check	0.099	11:44:21
Air Blank	0.000	11:44:51
Subject Test	0.000	11:45:17
Air Blank	RFI*	11:45:37
Air Blank	0.000	11:46:05

*RFI Detect

 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/06/2006

11:46:40

Standard Lot#: 536401I

Last Changed By: M. SLONEKER

#6559

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:47:28
Diagnostic Test	Pass	11:47:55
Air Blank	ABT*	11:48:03
Air Blank	IPS**	11:48:14

*Sequence Aborted

**Improper Sample

88 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number : 80-001858

07/06/2006

11:59:08

Standard Lot#: 536401I

Last Changed By: M. SLONEKER

#6559

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:00:04
Diagnostic Test	Pass	12:00:32
Air Blank	0.000	12:00:59
0.100 Cal Check	0.099	12:01:18
Air Blank	0.000	12:01:47
Subject Test	0.058*	12:04:56
Air Blank	0.000	12:05:27
Air Blank	0.000	12:05:55
Subject Test	XXX**	12:06:29
Air Blank	0.000	12:07:02

*Deficient Sample

**Invalid Sample

Begin new deprivation period

 #1123

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1211
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR JUSTICE FACILITY

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 099 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 536401 I
EXP 1-3-08

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number: 80-001858

07/06/2006

12:11:15

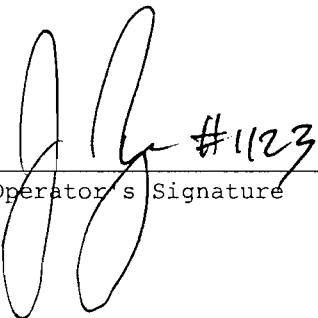
Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:11:42
Cal Check	0.099	12:12:01
Air Blank	0.000	12:12:31



Operator's Signature