

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GLEIMAN AGENCY GOODYEAR PD
DATE 7/20/2012 TIME 0416
INTOXILYZER SERIAL # 80-001558 LOCATION 10701 S ESTRELLA PKWY

- (☒) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (☒) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0. 099 AC.
- (☒) 6. Air blank completed.
- (☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000

Location: 10701 S ESTRELLA PKY

Serial Number: 80-001858

Core Version: 8105.48

07/20/2012

.04:16:08

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	04:16:39
0.100 Cal Check	0.099	04:17:00
Air Blank	0.000	04:17:30

Preventative Maintenance
Performed -- Timer Reset