EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPI	ECI	ALISTM. GUERNEY T AGENCY GOOS FEAR PA
DATE		7/80/13 TIME 0120
	IL.	YZER SERIAL # 80-00/858 LOCATION 10701 5 FETATUR PRO
(k)	1.	Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solutionAC. OR
゙゙゙゙゙゙゙゙゙゙゙゙		Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
(👌	2.	Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(7)	3.	Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is
1		made press ENTER.
(7)	4.	Air blank completed. Calibration check completed. Test results 0. 100 AC.
(/ P)		
Koa,)		Air blank completed.
(X)	7.	Remove printed record. Attach the record to the completed checklist.
SIGN	ATI	JRE

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000 Instrument Initialization 21:58:16 07/09/2013

INTOXILYZER BODD

Location: 19701 S ESTRELLA Serial Number: 80-001858 Core Version: 8105.48

07/10/2013

01:20:41

Standard Lot#: 34011100A1 Last Changed By: M. GUILBRULT #1144

QAS: M. GUILBAULT #1144

CCCOYEAR PO

Operator: M. GUILBAULT #1144

GCODYEAR PD

02100 Cal Check 0.100 Air Blank 0.000

01:21:41 01:22:11

Preventative Maintenance Performed -- Timer Reset

. 11.