

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST WILL AGENCY Goodyear PD
DATE 6/13/2009 TIME 19:24
INTOXILYZER SERIAL # 80-1858 LOCATION Goodyear PD

- W 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- W 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- W 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- W 4. Air blank completed.
- W 5. Calibration check completed. Test results 0. 101 AC.
- W 6. Air blank completed.
- W 7. Remove printed report. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

06/13/2009

19:24:05

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:24:32
Cal Check	0.101	19:24:51
Air Blank	0.000	19:25:20

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST WUC AGENCY Goodyear PD
DATE 6/13/2009 TIME 19:21
INTOXILYZER SERIAL # 80-1858 LOCATION Goodyear PD

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (M) 4. Air blank completed.
- (M) 5. Calibration check completed. Test results 0. 100 AC.
- (M) 6. Air blank completed.
- (M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

06/13/2009

19:20:46

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:21:14
Cal Check	0.100	19:21:32
Air Blank	0.000	19:22:02

Operator's Signature

804302I
EX 17 FEB 10

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST WILL AGENCY Georgetown PD
DATE 6/27/2009 TIME 1628
INTOXILYZER SERIAL # 80-1858 LOCATION Georgetown PD

- W* 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- W* 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- W* 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- W* 4. Air blank completed.
- W* 5. Calibration check completed. Test results 0. 100 AC.
- W* 6. Air blank completed.
- W* 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Jh*

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

06/27/2009

16:27:37

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	16:28:04
Cal Check	0.100	16:28:23
Air Blank	0.000	16:28:52

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST WILL AGENCY GOODYEAR PD
DATE 6/27/2009 TIME 1731
INTOXILYZER SERIAL # 80-1858 LOCATION GOODYEAR PD

(M) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (M) 1. Clock time check.
(M) 2. Date check.

OPERATIONAL TESTS

- (M) 1. Alcohol-free subject test result 0. 600 AC.
(M) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(M) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(M) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES (signature) NO _____

COMMENTS

SIGNATURE _____

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:31:47

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:32:36
Diagnostic Test	Pass	17:33:03
Air Blank	0.000	17:33:31
0.100 Cal Check	0.100	17:33:49
Air Blank	0.000	17:34:19
Subject Test	0.000	17:34:36
Air Blank	ABT*	17:34:42
Air Blank	0.000	17:35:10

*Sequence Aborted

HIT TEST BUTTON
DURING AIRBLANK
INSTEAD OF AT
WAIT PERIOD.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:35:21

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:36:18
Diagnostic Test	Pass	17:36:45
Air Blank	0.000	17:37:13
0.100 Cal Check	0.101	17:37:31
Air Blank	0.000	17:38:01
Subject Test	0.000	17:38:18
Air Blank	0.000	17:38:45
Five Minute Wait	ABT*	
Air Blank	0.000	17:39:23

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:39:29

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:40:12
Air Blank	0.000	17:40:40

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:40:53

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	17:41:39
Air Blank	0.000	17:42:07

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:42:14

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:43:16
Diagnostic Test	Pass	17:43:43
Air Blank	0.000	17:44:10
0.100 Cal Check	0.101	17:44:29
Air Blank	0.000	17:44:59
Subject Test	0.033*	17:48:07
Air Blank	0.000	17:48:39
Air Blank	0.000	17:49:06
Subject Test	RGE**	17:49:32
Air Blank	0.000	17:50:07

*Deficient Sample

**Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

06/27/2009

17:50:15

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT

#1144

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:51:11
Diagnostic Test	Pass	17:51:39
Air Blank	0.000	17:52:06
0.100 Cal Check	0.100	17:52:25
Air Blank	0.000	17:52:54
Subject Test	XXX*	17:53:07
Air Blank	0.000	17:53:41

*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

06/27/2009

17:54:03

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:54:31
Cal Check	0.100	17:54:49
Air Blank	0.000	17:55:19

Operator's Signature