

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GULLBAUT 1144 AGENCY GOODYEAR AD
DATE 6/7/2013 TIME 0058
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA Pkwy

(☒) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (☒) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(☒) 4. Air blank completed.
(☒) 5. Calibration check completed. Test results 0. 100 AC.
(☒) 6. Air blank completed.
(☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE M. GULLBAUT 1144

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-001858

Core Version: 8105.46

06/07/2013

00:58:27

Standard Lot#: 13712100A2

Last Changed By: M. GUILBAULT #1144

OAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	00:59:00
0.100 Cal Check	0.100	00:59:21
Air Blank	0.000	00:59:51

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-001858

Core Version: 8105.46

06/07/2013

00:58:27

Standard Lot#: 13712100A2

Last Changed By: M. GUILBAULT #1144

OAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	00:59:00
0.100 Cal Check	0.100	00:59:21
Air Blank	0.000	00:59:51

Preventative Maintenance
Performed -- Timer Reset