

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department  
DATE 3-16-2008 TIME 00:01  
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 400 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. .099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

03/16/2008

00:01:33

-----  
Standard Lot#: 706502I

Last Changed By: W. WILL #974

-----  
QAS: W. WILL #974

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:02:01
Cal Check	0.099	00:02:19
Air Blank	0.000	00:02:49

\_\_\_\_\_  
Operator's Signature

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.  
DATE 3-27-08 TIME \_\_\_\_\_  
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR PD

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 101 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

03/27/2008

13:17:01

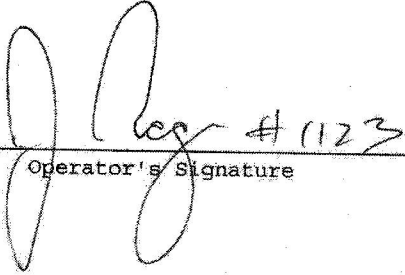
-----  
Standard Lot#: 7065021

Last Changed By: W. WILL #974

-----  
QAS: J. ROGERS #1123

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:17:28
Cal Check	0.101	13:17:47
Air Blank	0.000	13:18:17

  
-----  
Operator's Signature