

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. G. [Signature] 1144 AGENCY GOODYEAR PD
DATE 3/13/2010 TIME 1571
INTOXILYZER SERIAL # 80-001858 LOCATION 601

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution - 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 100 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8020
Location: GOODYEAR POLICE DEPT
Serial Number: 00-001856
03/13/2010

18:21:14

Standard Lot#: 562576
Last Change By: H. GUILBERT 01144

Off: H. GUILBERT 01144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:21:41
Cal Check	0.100	18:22:00
Air Blank	0.000	18:22:29


Officer's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GIBRISBAULT AGENCY GOODYEAR PD
DATE 3/19/2010 TIME 0059
INTOXILYZER SERIAL # 80-001858 LOCATION 600

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 101 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

03/19/2010

00:59:33


Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	01:00:00
Cal. Check	0.101	01:00:19
Air Blank	0.000	01:00:48

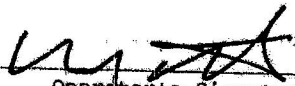

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GUSARAU AGENCY GOODYEAR PD
DATE 3/19/2010 TIME 0105
INTOXILYZER SERIAL # 80-001858 LOCATION GPD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed.
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE [Signature] 1144

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
03/19/2010

01:05:33

Standard Lot#: 562576
Last Changed By: M. GUILBAULT #1144
OAS: M. GUILBAULT #1144
GOODYEAR PD
Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST
DOB: 10/10/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	01:06:25
Diagnostic Test	Pass	01:06:53
Air Blank	0.000	01:07:20
0.100 Cal Check	0.102	01:07:39
Air Blank	0.000	01:08:08
Subject Test	0.000	01:08:26
Air Blank	0.000	01:08:54
Five Minute Wait	ABT*	
Air Blank	0.000	01:09:30

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
03/19/2010

01:10:15

Standard Lot#: 562576
Last Changed By: M. GUILBAULT #1144
QAS: M. GUILBAULT #1144
GOODYEAR PD
Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST
DOB: 10/10/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/10L	Time
Air Blank	IPS*	01:10:50
Air Blank	RFI**	01:10:59

*Improper Sample
**RFI Detect

✱

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
03/19/2010

01:11:31

Standard Lot#: 562576
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST
DOB: 10/10/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	01:12:30
Diagnostic Test	Pass	01:12:57
Air Blank	0.000	01:13:25
0.100 Cal Check	0.102	01:13:43
Air Blank	0.000	01:14:13
Subject Test	0.016*	01:17:21
Air Blank	0.000	01:17:53
Air Blank	0.000	01:18:21
Subject Test	XXX**	01:19:37
Air Blank	0.000	01:20:11

*Deficient Sample

**Invalid Sample

Begin new deprivation period

☆

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURMAULT AGENCY GOODYEAR PD
DATE 3/19/2010 TIME _____
INTOXILYZER SERIAL # 80-001858 LOCATION GPS

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 110 AC.
CR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

03/19/2010

01:21:38


Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

OAS: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	01:22:05
Cal Check	0.101	01:22:24
Air Blank	0.000	01:22:53

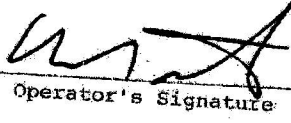

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Will AGENCY Goodyear PD
DATE 3-19-2010 TIME _____
INTOXILYZER SERIAL # 80-0018 SP LOCATION Goodyear PD

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution. 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
 4. Air blank completed.
 5. Calibration check completed. Test results 0. 101 AC.
 6. Air blank completed.
 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000
Instrument Initialization
10:35:40 03/19/2010

INTOXILYZER 8000
Location: GOODYEAR PD
Serial Number: 80-001858
Core Version: 8105.48
03/19/2010 10:47:13

Standard Lot#: 526576
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	10:47:52
0.100 Cal Check	0.101	10:48:13
Air Blank	0.000	10:48:42

Preventative Maintenance
Performed -- Timer Reset

Intoxilyzer 8000 Update Form

Lab Use Only	
Downloaded?	Yes No
Date Updated	
Spreadsheet?	Inst. QAS

Instrument Serial # 80-001858

Permanent Location GOODYEAR PD

COBRA Downloaded

Not Downloaded

Primary QAS Information

Name W. Will Badge 974

Agency GOODYEAR PD

Agency Address PO Box 5100
GOODYEAR AZ 85378

Work Phone 623 882 7093 Cell Phone 623 693 0072

Email hwill@GOODYEARAZ.GOV

Secondary QAS Information

Name M. Guilbault Badge 1144

Agency _____

Agency Address _____

Work Phone _____ Cell Phone _____

Email MGuilbault@GOODYEARAZ.GOV

* If this instrument is taken care of by additional QAS officers, please put their information on the back of this form.

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUTHAULT AGENCY GOODYEAR PD
DATE 3/27/2010 TIME 1573
INTOXILYZER SERIAL # 80-001858 LOCATION GPA

- (A) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (A) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (B) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (A) 4. Air blank completed.
- (A) 5. Calibration check completed. Test results 0. 101 AC.
- (A) 6. Air blank completed.
- (A) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____
[Signature]

INTOXILYZER 8000
Location: GOODYEAR PD
Serial Number: 80-001858
Core Version: 8105.48
03/27/2010

15:23:19

Standard Lot#: 526576
Last Changed By: W. WILL #974

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:23:50
0.100 Cal Check	0.101	15:24:11
Air Blank	0.000	15:24:40

Preventative Maintenance
Performed -- Timer Reset