

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Will AGENCY GOODYEAR PD
DATE 5/22/2008 TIME 1437
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR PD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 100 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE jh

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

05/22/2008

14:37:09

Standard Lot#: 808401G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|------------------|--------|----------|
| Air Blank | 0.000 | 14:38:03 |
| Diagnostic Test | Pass | 14:38:31 |
| Air Blank | 0.000 | 14:38:58 |
| 0.100 Cal Check | 0.100 | 14:39:17 |
| Air Blank | 0.000 | 14:39:46 |
| Subject Test | 0.000 | 14:40:30 |
| Air Blank | 0.000 | 14:40:57 |
| Five Minute Wait | ABT* | |
| Air Blank | 0.000 | 14:42:21 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
05/22/2008 14:43:18

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | IPS* | 14:43:55 |
| Air Blank | 0.000 | 14:44:23 |

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
05/22/2008 14:44:54

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 14:45:37 |
| Air Blank | 0.000 | 14:46:05 |

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
05/22/2008 14:46:27

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 14:47:16 |
| Diagnostic Test | Pass | 14:47:44 |
| Air Blank | 0.000 | 14:48:11 |
| 0.100 Cal Check | 0.101 | 14:48:30 |
| Air Blank | 0.000 | 14:48:59 |
| Subject Test | 0.025* | 14:52:08 |
| Air Blank | 0.000 | 14:52:40 |
| Air Blank | 0.000 | 14:53:07 |
| Subject Test | DEF** | 14:56:15 |
| Air Blank | 0.000 | 14:56:49 |

*Deficient Sample
**Deficient Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
05/22/2008 14:57:07

Standard Lot#: 808401G
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 14:57:58 |
| Diagnostic Test | Pass | 14:58:26 |
| Air Blank | 0.000 | 14:58:53 |
| 0.100 Cal Check | 0.101 | 14:59:12 |
| Air Blank | 0.000 | 14:59:42 |
| Subject Test | XXX* | 15:00:01 |
| Air Blank | 0.000 | 15:00:35 |

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Will AGENCY GOODYEAR PD
DATE 5/22/2008 TIME 1500
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR PD

- (W) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution 2 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (W) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (W) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (W) 4. Air blank completed.
- (W) 5. Calibration check completed. Test results 0. 100 AC.
- (W) 6. Air blank completed.
- (W) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

05/22/2008

15:00:56


Standard Lot#: 808401G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 15:01:23 |
| Cal Check | 0.100 | 15:01:42 |
| Air Blank | 0.000 | 15:02:12 |

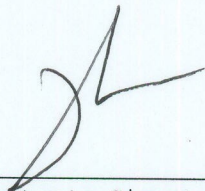

Operator's Signature