

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department
DATE 11/24/2008 TIME 1647
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

- (H) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (H) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (H) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (H) 4. Air blank completed.
- (H) 5. Calibration check completed. Test results 0. 100 AC.
- (H) 6. Air blank completed.
- (H) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

11/26/2008

16:46:45


Standard Lot#: 808401G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	16:47:13
Cal Check	0.100	16:47:31
Air Blank	0.000	16:48:01

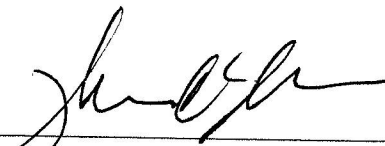

Operator's Signature