

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department
DATE 11/10/2013 TIME 1951
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 099 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

DPS Form Exh G-2 (Rev 05-01)

MTB

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/10/2013

19:51:10

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:51:45
0.100 Cal Check	0.099	19:52:06
Air Blank	0.000	19:52:36

Preventative Maintenance
Performed -- Timer Reset



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID: 4198
Part #: BAC105L100T
Cylinder Size: 105L
Lot Number: 32012100A1
Expiration: 1/1/2015

0.100 BAC (For use with breath alcohol testing instruments)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	260.5 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	Balance	BAC whichever is greater	

*NIST Standard Reference Material
Certification of NTRM Batch No. 091602
Nominal 210 µmol/mol Ethanol in Nitrogen
for ILMO Products Co., Jacksonville, IL

Jacob Matthes
Specialty Gas Lab Tech

12-03-12
Date

Distributed by: CMI Inc.
316 East Ninth Street
Owensboro, KY 42303
Phone 866-835-0690
www.alcoholtest.com

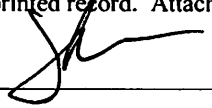


EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.C. Will #974 AGENCY Goodyear Police Department
DATE 11/10/2013 TIME 1955
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear Police Department

- (u) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 097 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/10/2013

19:55:26

Standard Lot#: 32012100A1

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:55:56
0.100 Cal Check	0.097	19:56:17
Air Blank	0.000	19:56:46

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. Ryan 1100 AGENCY GOODYEAR Police Dept
DATE 11-13-13 TIME 1412
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR Police Dept

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 10 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
11/13/2013 14:12:04

Standard Lot#: 32012100A1
Last Changed By: W. WILL #974

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 11/13/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:13:20
Diagnostic Test	Pass	14:13:55
Air Blank	0.000	14:14:23
0.100 Cal Check	0.099	14:14:44
Air Blank	0.000	14:15:13
Subject Test	0.000	14:15:34
Air Blank	0.000	14:16:01
Wait	ABT*	
Air Blank	0.000	14:16:39

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:17:10

Standard Lot#: 32012100A1

Last Changed By: W. WILL

#974

QAS: W. RYAN

#1100

GOODYEAR PD

Operator: W. RYAN

#1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/13/1959

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	14:17:58
Air Blank	IPS*	14:18:14

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:18:50

Standard Lot#: 32012100A1

Last Changed By: W. WILL

#974

QAS: W. RYAN

#1100

GOODYEAR PD

Operator: W. RYAN

#1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/13/1959

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	14:19:37
Air Blank	0.000	14:20:06

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:29:59

Standard Lot#: 32012100A1

Last Changed By: W. WILL

#974

QAS: W. RYAN

#1100

GOODYEAR PD

Operator: W. RYAN

#1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/13/1959

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:30:55
Diagnostic Test	Pass	14:31:30
Air Blank	0.000	14:31:58
0.100 Cal Check	0.099	14:32:19
Air Blank	0.000	14:32:48
Subject Test	0.008*	14:35:55
Air Blank	0.000	14:36:23
Air Blank	0.000	14:36:50
Subject Test	XXX**	14:37:15
Air Blank	0.000	14:37:49

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:20:52

Standard Lot#: 32012100A1

Last Changed By: W. WILL

#974

QAS: W. RYAN

#1100

GOODYEAR PD

Operator: W. RYAN

#1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/13/1959

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:22:13
Diagnostic Test	Pass	14:22:48
Air Blank	0.000	14:23:15
0.100 Cal Check	0.099	14:23:36
Air Blank	0.000	14:24:06
Subject Test	0.017*	14:27:13
Air Blank	0.000	14:27:44
Air Blank	0.000	14:28:11
Subject Test	RGE**	14:28:29
Air Blank	0.000	14:29:09

*Deficient Sample

**Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:39:39

Standard Lot#: 32012100A1

Last Changed By: W. WILL #974

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:40:10
0.100 Cal Check	0.098	14:40:31
Air Blank	0.000	14:41:01

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR Police Dept
DATE 11-13-13 TIME 1907
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR Police Dept

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (☒) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0. 098 AC.
- (☒) 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

11/13/2013

14:07:59

Standard Lot#: 32012100A1

Last Changed By: W. WILL #974

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:08:35
0.100 Cal Check	0.098	14:08:56
Air Blank	0.000	14:09:26

Preventative Maintenance

Performed -- Timer Reset