

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUILBERT 1144 AGENCY GOODYEAR PD
DATE 10/9/2012 TIME 2353
INTOXILYZER SERIAL # RD-001858 LOCATION 10901 S ESTERLA PKWY

- (~~1~~) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (~~2~~) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (~~3~~) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (~~4~~) 4. Air blank completed.
- (~~5~~) 5. Calibration check completed. Test results 0. 100 AC.
- (~~6~~) 6. Air blank completed.
- (~~7~~) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____


INTOXILYZER 8000 16701
Location: 10901 S ESTRELLA PKW
Serial Number: 80-001858
Core Version: 8105.48
10/09/2012

23:53:00

Standard Lot#: 34011100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	23:53:29
0.100 Cal Check	0.100	23:53:50
Air Blank	0.000	23:54:19

Preventative Maintenance
Performed -- Timer Reset