EXHIBIT G-4 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K- MIGAZATO 1/ AGENCY GOODY ZAR
DATE 5/6/2020 TIME 23:27
INTOXILYZER SERIAL# 80-00/858 LOCATION 1/ N 145 TH AVE
() 1. Display Reads "PUSH BUTTON TO START"
DIAGNOSTIC TESTS (1) 1. Clock time check. (1) 2. Date check.
OPERATIONAL TESTS () 1. Alcohol-free subject test result 0. OOO AC. () 2. Error recognition logic system functioning. Not a Successfully Completed Test Sequence printed () 3. Proper sample recognition system. Not a Successfully Completed Test Sequence printed Deficient sample printed. () 4. Standard Calibration Check standard 0. AC. Result 0. O G AC.
Instrument is operating properly and accurately. YES NO
COMMENTS I FINISHED ALL STEPS EXCEPT CALIBRATION CHECK DUE TO
MAGNETIC STRIP NOT PRESDING MY CARD. I HAD IT SENT INTO PPS
D GET FIXED. THEY FIXED USUES AND PETUNOSO IT-AM
ADVISED I MESOSO TO COMPLETE 3/DAM CHEYE
GIGNATURE 139

DPS Form Exh G-4 (Rev 05-01)

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/01/2020 23:27:36

Standard Lot#: 10818100A1
Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 203

15 Minute Deprivation Period? Yes

Time Test g/210L 0.000 23:29:38 Air Blank 23:30:13 Diagnostic Test Pass Air Blank 0.000 23:30:41 0.100 Cal Check 0.100 23:31:01 23:31:31 Air Blank 0.000 Subject Test 0.000 23:31:52 Air Blank ABT* 23:32:04 Air Blank 0.000 23:32:32

^{*}Sequence Aborted

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/01/2020 23:32:49

Standard Lot#: 10818100A1 Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 203

15 Minute Deprivation Period? Yes

 Test
 g/210L
 Time

 Air Blank
 IPS*
 23:33:43

 Air Blank
 0.000
 23:34:11

^{*}Improper Sample

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/01/2020 23:35:50

Standard Lot#: 10818100A1
Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139 GOODYEAR

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 203

15 Minute Deprivation Period? Yes

Test	g/210L	Time		
Air Blank	RFI*	23:36:44		
Air Blank	0.000	23:37:13		

^{*}RFI Detect

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/01/2020 23:37:29

Standard Lot#: 10818100A1
Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 203

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:38:40
Diagnostic Test	Pass	23:39:15
Air Blank	0.000	23:39:42
0.100 Cal Check	0.101	23:40:03
Air Blank	0.000	23:40:33
Subject Test	0.012*	23:43:40
Air Blank	0.000	23:44:09
Air Blank	0.000	23:44:37
Subject Test	XXX**	23:45:01
Air Blank	0.000	23:45:35

^{*}Deficient Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

^{**}Invalid Sample

$EXHIBIT~G-2 \\ THIS REPORT PREPARED PURSUANT~TO~DUTY~IMPOSED~BY~A.A.C.~R13-10-104~(A) \\$

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KM (4A-ZATO) AGENCY (2009 SM
DATE 5/6/2020 TIME 21:31
INTOXILYZER SERIAL # 80-00/858 LOCATION // N 145 TH AVE
1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution AC.
Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
 Intoxilyzer 8000 display reads "PUSH BUTTON TO START" Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed. 5. Calibration check completed. Test results 0. 099 AC. 6. Air blank completed.
7. Remove printed/record. Attach the record to the completed checklist.
SIGNATURE 139
DPS Form Exh G-2 (Rev 05-01)

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/06/2020 21:31:49

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:32:19
0.100 Cal Check	0.099	21:32:40
Air Blank	0.000	21:33:09

Preventative Maintenance Performed -- Timer Reset



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790 217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID:

11912

Part #:

BAC105L100T

Cylinder Size:

105L

Lot Number:

07919100A1

Expiration:

6/5/2021

0, 100 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Reported

Analytical

Component:

Concentration:

Accuracy

(U, k=2):

Method:

Ethanol Nitrogen 260 ppm Balance

+/-0.002 BAC(G/210L) NDIR

CMI Inc.

[5.2 ppm]

Distributed by:

316 East Ninth Street Owensboro, KY 42303 Phone 866-835-0690

www.alcoholtest.com

*NIST Traceable Reference Material Cylinder No. SGAL1091 / Job No. 13029 Certified 373.4 µmol/mol Ethanol in Nitrogen Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Specialry Gas Lab Tech

04-26-19

Accreditation #61895

The calibration results within this certificate were obtained using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service.



Safety Data Sheet

Ethanol in Nitrogen

www.ilmoproducts.com

Section 1: Product and Company Identification

ILMO Products Company 7 Eastgate Drive, Jacksonville, Illinois 62650 217-245-2183 800-424-9300 (Chemtrec)

Fax 217-243-7634 E-mail: info@ilmoproducts.com Web: www.ilmoproducts.com

Product Code: Ethanol in Nitrogen Part Number: BAC

Section 2: Hazards Identification



Hazard Classification: Gases Under Pressure

Hazard Statements: Contains gas under pressure; may explode if heated

Precautionary Statements

Storage: Protect from sunlight. Store in well-ventilated place.

Section 3: Composition/Information on Ingredients

	CAS#	Concentration		
Ethanol	64-17-5	5-500 ppm		
Nitrogen	7727-37-9	Balance		

ILMO Products Company Generated by the SDS Manager from AsteRisk, LLC: All Rights Reserved

page 1 of 6 Generated: 02/08/2017 10:18:45

	Methods for Cleanup	Other Information
Ethanol	Small spills: Absorb with sand or other noncombustible material. Collect spilled material in appropriate container for disposal. Large spills: Dike for later disposal.	Not available
Nitrogen	N/A	N/A

Section 7: Handling and Storage

	Handling	Storage		
Ethanol	Store and use with adequate ventilation. Firmly secure cylinders upright to keep them from falling or being knocked over. Serve valve protection cap firmly in place by hand. Store only where temperature will not exceed 125°F (52°C). Store full and empty cylinders separately. Use a first-in, first-out inventory system to prevent storing full Cylinders for long periods.	Do not get liquid in eyes, on skin, or clothing. Protect cylinders from damage. Use a suitable hand truck to move cylinders; do not drag, roll, side, or drop. Open valve slowly Close cylinder valve after each use; keep closed even when empty. If valve is hard to open, discontinue use and contact your supplier.		
Nitrogen	Store and handle in accordance with all current regulations and standards. Subject to storage regulations: U.S. OSHA 29 CFR 1910.101.	Keep separated from incompatible substances.		

Section 8: Exposure Controls/Personal Protection

	Exposure Guidelines
Ethanol	ETHYL ALCOHOL, 100%: ETHYL ALCOHOL (ETHANOL): 1000 ppm (1900 mg/m3) OSHA TWA 1000 ppm ACGIH TWA 1000 ppm (1900 mg/m3) NIOSH recommended TWA 10 hour(s)
Allibranes	(1900 mg/m3) NIOSH recommended I WA 10 hour(S) NITROGEN, COMPRESSED, CAS, NITROGEN, ASCIN (simple contrained)

Engineering Controls

	Eye Protection	Skin Protection	Respiratory Protection			
Ethanol	Wear splash resistant safety goggles. Provide an emergency eye wash fountain and quick drench shower in the immediate work area.	Wear appropriate chemical resistant clothing.	Any supplied-air respirator with full facepiece and operated in a pressure-demand or other positive- pressure mode in combination with a separate escape supply.			
Nitrogen	Eye protection not required, but recommended.	Protective clothing is not required	Respiratory protection may be needed for frequent or			

- General Hygiene considerations

 Avoid breathing vapor or mist
 Avoid contact with eyes and skin
 Wash thoroughly after handling and before eating or drinking

Section 9: Physical and Chemical Properties

		Appearance	Color	Change in Appearance	Physical Form	Odor	Taste
Ethanol Li	iquid	Clear	Colorless	N/A	Volatile liquid	Pleasant odor	Burning taste
Nitrogen G	Sas	Clear	Colorless	N/A	Gas	Odorless	Tasteless

			,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,	Coefficien		emperature		Limits	Limits	-xpicaire
Ethanol	55 F (13 (CC)	C) IB		Not availab	le 6	85 F (363 C)		0.19	0.033	
Nitrogen	Not flamn	nable No	available	Not availab	e N	lonflammable		Nonflammable	Nonflam	mable
	Boiling Point	Freezing Point	Vapor Pressure	Vapor	Specific Gravity	Water Solubility	pH	Odor Threshold	Evaporation Rate	Viscosity

	Chemical Substance	Chemical Family	Trade Names
Ethanol	ETHYL ALCOHOL, 100%	hydroxyls, aliphatic, alcohols, aliphatic	ETHANOL; ETHYL, ALCOHOL; A., ALCOHOL; ALCOHOL ANHYDROUS; ALGRAIN; ANHYDROUS; ALGRAIN; ANHYDROL; Absolute alcohoi; Anhydrous elhand; Ethand denatured; Fermentation alcohoi, Grain alcohoi; 1-Hydroxyethane, Methyl carthroid; Ethyl alcohol anhydrous; Absolute ethanoi; Denatured ethanoi; ETHYL HYDRATE; ETHYL HYDROXIDE; JAYSOL; TECSOL; STCC 4999159; UN 1170; CZH6O
Nitrogen	NITROGEN, COMPRESSED GAS	Inorganic gases	DIATOMIC NITROGEN; DINITROGEN; NITROGEN; NITROGEN-14; NITROGEN GAS; UN 1066; N2

Section 4: First Aid Measures

	Skin Contact	Eye Contact	Ingestion	Inhalation	Note to Physicians
Ethanoi	Wash skin with soap and water for at least 15 minutes while removing contaminated ciothing and shoes. Get medical attention, if needed. Thoroughly clean and dry contaminated ciothing and shoes before reuse.	Flush eyes with plenty of water for at least 15 minutes. Then get immediate medical attention.	Contact local poison control center or physician immediately. Nover make an unconscious person vomit or drink fluids. When vomiting occurs, keep head lower than hips to help prevent aspiration. If person is unconscious, turn head to side. Get medical attention immediately.	If adverse effects occur, remove to uncontaminated area. Give artificial respiration if not breathing. Get immediate medical attention.	For ingestion consider gastric lavage.
Vitrogen	Wash exposed skin with soap and water.	Flush eyes with plenty of water.	If a large amount is swallowed, get medical attention.	If adverse effects occur, remove to uncontaminated area. Give artificial respiration if not breathing. If breathing is difficult, oxygen should be administered by qualified personnel. Get immediate medical attention.	For inhalation, consider oxygen.

Section 5: Fire Fighting Measures

4.2	Suitable Extinguishing Media	Products of Combustion	Protection of Firefighters
Ethanol	Alcohol resistant foam, carbon dloxide, regular dry chemical, water, alcohol resistant foam Large fires: Use alcohol-resistant foam or flood with fine water spray.	Carbon monoxide, carbon dioxide, and toxic and irritating furnes	 Any supplicatir respirator with full facepiece and operated in a pressure-demand or other positive-pressure model in combination with a separate escape supplie. Any supplied-air respirator with full facepiece and operated in a pressure-demand or other positive-pressure model in combination with a separate escape supplied.
Nitrogen	Non-flammable. Use suitable extinguishing media for surrounding fire. Cylinders may rupture or explode if exposed to heat.	Non-flammable	 Respiratory protection may be needed for frequent or heavy exposure.

Section 6: Accidental Release Measures

	Personal Precautions	Environmental Precautions	Methods for Containment
Ethanol	Keep unnecessary people away, isolate hazard area and deny entry. Stay upwind and keep out of low areas.	Avoid heat, flames, sparks and other sources of ignition.	Stop leak if possible without personal risk. Reduce vapors with water spray. Remove sources of ignition.
Nitrogen	Keep unnecessary people away, isolate hazard area and deny entry. Stay upwind and keep out of low areas.	No significant effects from contamination expected.	Stop leak if possible without personal risk.

ILMO Products Company Generated by the SDS Manager from AsteRisk, LLC. All Rights Reserved

page 2 of 6 Generated: 02/08/2017 10:18:45

	Boiling Point	Freezing Point	Vapor Pressure	Vapor Density	Specific Gravity	Water Solubility	рН	Odor Threshold	Evaporation Rate	Viscosity
Ethanol	172 F (78 C)	-179 F (- 117 C)	40 mmHg @ 19 C	1.59 (Air=1)	0.7893	Soluble	Not available	5-10 ppm	1.4 (carbon tetrachloride=1)	1.17 mPa.s (1.17 centipoises @ 20 C; 1.074 mPa.s (1.074 centipoises @ 25 C
Nitrogen	-321 F (-196 C)	-346 F (- 210 C)	760 mmHg @ -196 C	0.967 (Air=1)	Not applicable	1.6% @ 20 C	Not applicable	Not available	Not applicable	0.01787 cP @ 27 C

	Weight	Formula	Density	Gallon	Volatility by Volume	Volatility	Solvent Solubility
Ethanol	46.07	C-H3-C-H2-O- H	Not available	Not available	Not available	1	Soluble: Benzene, ether, acetone, chloroform, methanol, organic solvents
Nitrogen	28.0134	N2	1.2506 g/L	Not available	100%	1	Soluble: Liquid ammonia

Section 10: Stability and Reactivity

	Stability	Conditions to Avoid	Incompatible Materials
Ethanol	Stable at normal temperatures and pressure.	Stable at normal temperatures and pressure.	Halo carbons, metals, metal salts, oxidizing materials, halogens, peroxides, acids, metal oxides, bases, combustible materials
Nitrogen	Stable at normal temperatures and pressure.	Stable at normal temperatures and pressure.	Metals, oxidizing materials

	Hazardous Decomposition Products	Possibility of Hazardous Reactions
Ethanol	Oxides of carbon	Will not polymerize.
Nitrogen	Oxides of nitrogen	Will not polymerize.

Section 11: Toxicology Information

	Oral LD50	Dermal LD50	Inhalation
Ethanol	7 gm/kg oral- rat LD50	LD50 (dermal, rabbit): greater than 15800 mg/kg (cited as greater than 20 mL/kg); at 20 mL/kg, 1/4 rabbits died	Irritation, difficulty breathing, headache, drowsiness, symptoms of drunkenness
Nitrogen	Not available	Not available	Nausea, vomiting, difficulty breathing, headache, drowsiness, dizziness, tingling sensation, loss of coordination, convulsions coma

	Eye irritation	Skin Irritation	Sensitization
Ethanol	Irritation, tearing	Mild irritation, rash	Respiratory tract irritation, skin irritation, eye irritation, liver damage, central nervous system depression
Nitrogen	Contact with rapidly expanding gas may cause burns or frostbite	No information on significant adverse effects	Difficulty breathing

	Carcinogenicity	Mutagenicity	Reproductive Effects	Developmental Effects
Ethanol	NTP: Known Human Carcinogen (Alcoholic beverages); IARC: Human Sufficient Evidence, Group 1 (Alcoholic beverages), Animal Inadequate Evidence; ACGIH: A4-Not Classifiable as a Human Carcinogen	Available.	Available.	No data
Nitrogen	Not hazardous	Not available	Not available	No data

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KM (YAZITO 1138 AGENCY (TOODY ZAR
DATE 5(30/20 TIME 21:13
INTOXILYZER SERIAL # 80-80 858 LOCATION UN 145 TH AWE
1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution OR
Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER. 4. Air blank completed. 5. Calibration check completed. Test results 0AC. 6. Air blank completed. 7. Remove printed record. Attach the record to the completed checklist.
SIGNATURE \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \

DPS Form Exh G-2 (Rev 05-01)

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

05/30/2020 21:13:19

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:13:54
0.100 Cal Check	0.099	21:14:14
Air Blank	0.000	21:14:44

Preventative Maintenance Performed -- Timer Reset