EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST TEFF ROGERS AGENCY	GOODYEAR P.D.
DATE 9-16.07 TIME	1809
INTOXILYZER SERIAL# 80-001858 LOCATIO	ON GOODYEAR PD
() 1. Ensure that gas tank is attached to instrument and contains a s	standard alcohol concentration solution
Pour a standard alcohol concentration solution	
Intoxilyzer 8000 display reads "PUSH BUTTON TO START	"
 3. Go to the "Control Testing Menu". Select "D" for dry contro made press ENTER. 4. Air blank completed. 5. Calibration check completed. Test results 0. 	l test or "W" for wet control test. After selection is
made press ENTER.	-53640
5. Calibration check completed. Test results 0.	AC. LOT 0 1-3-08
() O. The blank complete.	
7. Remove printed record. Attach the record to the completed c	necklist.
SIGNATURE #112	3
DPS Form Exh G-2 (Rev 05-01)	

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

18:09:12 09/16/2007

......

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test g/210L Time Air Blank 0.000 18:09:39
Cal Check 0.099 18:09:58
Air Blank 0.000 18:10:28

#1123 Operator's Signature

EXHIBIT G-4 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST TENT ROCERS DATE 9-16-07 INTOXILYZER SERIAL # 80-001858	AGENCY GODDYEARS PD
DATE 9-16-07	TIME 1812
INTOXILYZER SERIAL # 80 - 001858	LOCATION GOODYEAR PD
() 1. Display Reads "PUSH BUTTON TO START"	
DIAGNOSTIC TESTS (V) X. Clock time check. (V) 2. Date check.	LOJ 536401
OPERATIONAL TESTS () 2. Alcohol-free subject test result 0. O AC () 2. Error recognition logic system functioning. Not a Successfully Completed Test Sequence Not a Successfully Completed Test Sequence Deficient sample printed. () 4. Standard Calibration Check standard 0. 100	te printed te printed AC. Result 0. 098 AC.
Instrument is operating properly and accurately. YES 🔀	_ NO
COMMENTS	
SIGNATURE DPS Form Exh G-4 (Rev 05-01)	1123
\bigvee	

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

09/16/2007 18:12:21

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Operator: J. ROGERS #1123 GOODYEAR PD

Subject: TEST, TEST DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:13:30
Diagnostic Test	Pass	18:13:57
Air Blank	0.000	18:14:24
0.100 Cal Check	0.099	18:14:43
Air Blank	0.000	18:15:13
Subject Test 🤇	0.000	18:15:31
Air Blank	0.000	18:15:58
Five Minute Wait	ABT*	
Air Blank	0.000	18:16:37

^{*}Sequence Aborted

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

09/16/2007 18:17:07

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

#1123

QAS: J. ROGERS GOODYEAR PD

Operator: J. ROGERS #1123

GOODYEAR PD

Subject: TEST, TEST DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test g/210L Time

Air Blank IPS* 18:17:49

Air Blank 0.000 18:18:18

^{*}Improper Sample

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

09/16/2007 18:18:39

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Operator: J. ROGERS #1123

GOODYEAR PD

Subject: TEST, TEST DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

 Test
 g/210L
 Time

 Air Blank
 RFI*
 18:19:18

 Air Blank
 0.000
 18:19:47

*RFI Detect

USED NEXTER DIC

Not a Successfully Completed Test Sequence

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

09/16/2007 18:20:20

Standard Lot#: 5364011

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Operator: J. ROGERS #1123

GOODYEAR PD

Subject: TEST, TEST DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:21:08
Diagnostic Test	Pass	18:21:36
Air Blank	0.000	18:22:03
0.100 Cal Check	0.100	18:22:22
Air Blank	0.000	18:22:52
Subject Test	0.024*	18:26:00
Air Blank	0.000	18:26:30
Air Blank	0.000	18:26:57
Subject Test	XXX**	18:27:52
Air Blank	0.000	18:28:25

^{*}Deficient Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

^{**}Invalid Sample

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST ROGERS DATE 9-16-07	GENCY GOODYEA	R.P.D.
DATE 9-16-07	ME_ 1829	
INTOXILYZER SERIAL # 80 -001858	OCATION GOODYE	AR P.D.
(1. Ensure that gas tank is attached to instrument and co	tains a standard alcohol concer	-
Pour a standard alcohol concentration solution simulator. Ensure that a tight seal has been made. 0.2°C	irn on the simulator and allow	
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO 3. Go to the "Control Testing Menu". Select "D" for control Testing Menu".	y control test or "W" for wet co	
made press ENTER. (4. Air blank completed.	· ·	LOT 5364017 EXP 1-308
 4. All blank completed. 5. Calibration check completed. Test results 0. 6. Air blank completed. 7. Remove printed record. Attach the record to the content of the con	AC.	EXP 1-3-08
7. Remove printed record. Attach the record to the con	pleted checklist.	
SIGNATURE LOS #112	3	
DPS Form Exh G-2 (Rev 05-01)		

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

09/16/2007 18:29:06

Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 18:29:34
Cal Check 0.098 18:29:52
Air Blank 0.000 18:30:22

Operator's Signature