

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GILBERT 1144 AGENCY GOOD YEAR PA  
DATE 9/6/2012 TIME 0005  
INTOXILYZER SERIAL # 80-00185B LOCATION 10701 S ESTRELA PRWY

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 100 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 10701 S ESTRELLA PKY

Serial Number: 80-001858

Core Version: 8105.48

09/06/2012

00:05:26

A

-----  
Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144

GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144

GOODYEAR PD  
-----

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 00:05:56 |
| 0.100 Cal Check | 0.100  | 00:06:17 |
| Air Blank       | 0.000  | 00:06:47 |

Preventative Maintenance

Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GLENN BART 1144 AGENCY GOODTEAM PD  
DATE 9/6/2012 TIME 0006  
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PRWY

☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ☒ 1. Clock time check.  
☒ 2. Date check.

OPERATIONAL TESTS

- ☒ 1. Alcohol-free subject test result 0. 000 AC.  
☒ 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
☒ 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
☒ 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE [Signature] 1144

INTOXILYZER 8000  
Location: 10701 S ESTRELLA PKY  
Serial Number: 80-001858  
Core Version: 8105.48  
09/06/2012

00:06:56

Standard Lot#: 34011100A1  
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144  
GOODYEAR PD

Operator: M. GUILBAULT #1144  
GOODYEAR PD

Subject: TEST,TEST  
DOB: 01/01/1990  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 00:07:40 |
| Diagnostic Test | Pass   | 00:08:15 |
| Air Blank       | 0.000  | 00:08:43 |
| 0.100 Cal Check | 0.100  | 00:09:04 |
| Air Blank       | 0.000  | 00:09:33 |
| Subject Test    | 0.000  | 00:09:47 |
| Air Blank       | IPS*   | 00:09:54 |
| Air Blank       | RFI**  | 00:10:03 |

\*Improper Sample  
\*\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 10701 S ESTRELLA PKY

Serial Number: 80-001858

Core Version: 8105.48

09/06/2012

00:10:07

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 00:10:53 |
| Diagnostic Test | Pass   | 00:11:28 |
| Air Blank       | 0.000  | 00:11:55 |
| 0.100 Cal Check | 0.100  | 00:12:16 |
| Air Blank       | 0.000  | 00:12:45 |
| Subject Test    | 0.141* | 00:15:53 |
| Air Blank       | 0.000  | 00:16:26 |
| Air Blank       | 0.000  | 00:16:53 |
| Subject Test    | XXX**  | 00:17:09 |
| Air Blank       | 0.000  | 00:17:43 |

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 10701 S ESTRELLA PKY

Serial Number: 80-001858

Core Version: 8105.48

09/06/2012

00:17:48

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT

#1144

QAS: M. GUILBAULT

#1144

GOODYEAR PD

Operator: M. GUILBAULT

#1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 00:18:31 |
| Diagnostic Test | Pass   | 00:19:06 |
| Air Blank       | ABT*   | 00:19:13 |
| Air Blank       | 0.000  | 00:19:42 |

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GILBERT 1144 AGENCY GOODYEAR PD  
DATE 9/6/2012 TIME 0019  
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 099 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000  
Location: 10701 S ESTRELLA PKY  
Serial Number: 80-001858  
Core Version: 8105.48  
09/06/2012

00:19:54

-----  
Standard Lot#: 34011100A1  
Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 00:20:23 |
| 0.100 Cal Check | 0.099  | 00:20:43 |
| Air Blank       | 0.000  | 00:21:13 |

Preventative Maintenance  
Performed -- Timer Reset