

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazawa AGENCY GOODYEAR  
DATE 10/31/20 TIME 0505  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE


() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS  
() 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS  
() 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE 

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

10/31/2020

05:05:00

---

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 195

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	05:06:07
Diagnostic Test	Pass	05:06:42
Air Blank	0.000	05:07:09
0.100 Cal Check	0.099	05:07:30
Air Blank	0.000	05:08:00
Subject Test	0.000	05:08:17
Air Blank	0.000	05:08:44
Wait	ABT*	
Air Blank	0.000	05:09:26

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

10/31/2020

05:11:33

---

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 195

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	05:12:17
Air Blank	0.000	05:12:45

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

10/31/2020

05:12:48

---

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 195

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	05:13:26
Air Blank	0.000	05:13:54

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

10/31/2020

05:13:58

---

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M

Weight: 195

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	05:14:54
Diagnostic Test	Pass	05:15:29
Air Blank	0.000	05:15:56
0.100 Cal Check	0.098	05:16:17
Air Blank	0.000	05:16:48
Subject Test	0.429*	05:19:55
Air Blank	0.000	05:20:34
Air Blank	0.000	05:21:01
Subject Test	XXX**	05:21:14
Air Blank	0.000	05:21:48

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K. MIYAZATO AGENCY GOODYEAR  
DATE 10/31/20 TIME 0523  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0.098 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K. Miyazato 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

10/31/2020

05:23:16

-----  
Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

-----  
QAS: K. MIYAZATO #1139

GOODYEAR PD

-----  
Operator: K. MIYAZATO #1139

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	05:23:46
0.100 Cal Check	0.098	05:24:07
Air Blank	0.000	05:24:37

Preventative Maintenance  
Performed -- Timer Reset