

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K. MIYAZATO 1139 AGENCY Goodyear PD
DATE 2/28/21 TIME 3:39
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

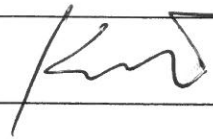
- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE  1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

02/28/2021

03:44:57

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	03:46:03
Diagnostic Test	Pass	03:46:38
Air Blank	IPS*	03:46:48
Air Blank	0.000	03:47:16

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
02/28/2021

03:39:56

Standard Lot#: 07919100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	03:41:25
Diagnostic Test	Pass	03:42:00
Air Blank	0.000	03:42:28
0.100 Cal Check	0.098	03:42:48
Air Blank	0.000	03:43:19
Subject Test	0.000	03:43:33
Air Blank	ABT*	03:43:44
Air Blank	0.000	03:44:12

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

02/28/2021

03:48:08

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	03:49:19
Air Blank	0.000	03:49:48

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
02/28/2021

03:50:03

Standard Lot#: 07919100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	03:51:02
Diagnostic Test	Pass	03:51:37
Air Blank	0.000	03:52:05
0.100 Cal Check	0.098	03:52:26
Air Blank	0.000	03:52:56
Subject Test	0.249*	03:56:03
Air Blank	0.000	03:56:37
Air Blank	0.000	03:57:04
Subject Test	0.064	03:57:22
Air Blank	IPS**	03:57:29
Air Blank	0.000	03:57:59

*Deficient Sample

**Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K MIYAZATO 1139 AGENCY GOODYEAR
DATE 2/28/21 TIME _____
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution ~~0.100~~ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

02/28/2021

03:59:53

Standard Lot#: 07919100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	04:00:23
0.100 Cal Check	0.098	04:00:44
Air Blank	0.000	04:01:14

Preventative Maintenance
Performed -- Timer Reset