

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazato AGENCY Goodyear PD  
DATE 7/30/21 TIME 1908  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. \_\_\_\_\_ AC. Result 0. \_\_\_\_\_ AC.

Instrument is operating properly and accurately. YES  NO \_\_\_\_\_

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE K Miyazato 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

07/30/2021

19:08:35

---

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO

#1139

---

QAS: K. MIYAZATO

#1139

GOODYEAR PD

---

Operator: K. MIYAZATO

#1139

GOODYEAR

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	19:10:03
Diagnostic Test	Pass	19:10:38
Air Blank	0.000	19:11:05
0.100 Cal Check	0.098	19:11:26
Air Blank	0.000	19:11:56
Subject Test	0.000	19:12:12
Air Blank	ABT*	19:12:18
Air Blank	0.000	19:12:47

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

07/30/2021

19:12:58

---

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO

#1139

---

QAS: K. MIYAZATO

#1139

GOODYEAR PD

---

Operator: K. MIYAZATO

#1139

GOODYEAR

---

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	19:13:57
Air Blank	0.000	19:14:25

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

07/30/2021

19:14:28

---

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	19:15:01
Air Blank	0.000	19:15:30

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
07/30/2021

19:15:32

---

Standard Lot#: 27220100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	19:16:19
Diagnostic Test	Pass	19:16:54
Air Blank	0.000	19:17:22
0.100 Cal Check	0.098	19:17:43
Air Blank	0.000	19:18:13
Subject Test	0.157*	19:21:20
Air Blank	0.000	19:21:53
Air Blank	0.000	19:22:20
Subject Test	XXX**	19:22:36
Air Blank	0.000	19:23:09

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazaki AGENCY Goodyear P.D.  
DATE 7/30/21 TIME 1923  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 098 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

07/30/2021

19:23:40

-----  
Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO #1139

-----  
QAS: K. MIYAZATO #1139

GOODYEAR PD

-----  
Operator: K. MIYAZATO #1139

GOODYEAR PD

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      19:24:09  
0.100 Cal Check                0.098                      19:24:30  
Air Blank                      0.000                      19:25:00

Preventative Maintenance  
Performed -- Timer Reset