

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K MIYAZATO 1139 AGENCY GOODYEAR PD  
DATE 10/31/2021 TIME 0331  
INTOXILYZER SERIAL # 80-000312 LOCATION // N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE K Miyazato 1139

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
10/31/2021 03:31:00

---

Standard Lot#: 27220100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	03:31:56
Diagnostic Test	Pass	03:32:31
Air Blank	0.000	03:32:59
0.100 Cal Check	0.099	03:33:19
Air Blank	0.000	03:33:49
Subject Test	0.000	03:34:02
Air Blank	ABT*	03:34:10
Air Blank	0.000	03:34:38

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
10/31/2021 03:34:41

---

Standard Lot#: 27220100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	03:35:15
Air Blank	0.000	03:35:44

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
10/31/2021 03:35:47

---

Standard Lot#: 27220100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	03:36:38
Air Blank	0.000	03:37:07

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
10/31/2021

03:37:11

---

Standard Lot#: 27220100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	03:37:58
Diagnostic Test	Pass	03:38:33
Air Blank	0.000	03:39:01
0.100 Cal Check	0.099	03:39:22
Air Blank	0.000	03:39:52
Subject Test	0.111*	03:42:59
Air Blank	0.000	03:43:33
Air Blank	0.000	03:44:00
Subject Test	XXX**	03:44:15
Air Blank	0.000	03:44:48

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazaki 1139 AGENCY GOODYEAR PD  
DATE 10/31/21 TIME ~~0845~~ 0345  
INTOXILYZER SERIAL # 80-000312 LOCATION // N 145 TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
() 4. Air blank completed.  
() 5. Calibration check completed. Test results 0.098 AC.  
() 6. Air blank completed.  
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazaki 1139

