

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY GOODYEAR PD
DATE 11/30/21 TIME 18:32
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE K Miyazato 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
11/30/2021 18:32:39

Standard Lot#: 27220100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 18:33:26 |
| Diagnostic Test | Pass | 18:34:01 |
| Air Blank | 0.000 | 18:34:28 |
| 0.100 Cal Check | 0.099 | 18:34:49 |
| Air Blank | 0.000 | 18:35:19 |
| Subject Test | 0.000 | 18:35:34 |
| Air Blank | ABT* | 18:35:40 |
| Air Blank | 0.000 | 18:36:09 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
11/30/2021 18:36:13

Standard Lot#: 27220100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP, TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | IPS* | 18:36:42 |
| Air Blank | 0.000 | 18:37:10 |

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

11/30/2021

18:37:15

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 18:37:50 |
| Air Blank | 0.000 | 18:38:18 |

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

11/30/2021

18:38:30

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 18:39:15 |
| Diagnostic Test | Pass | 18:39:49 |
| Air Blank | 0.000 | 18:40:17 |
| 0.100 Cal Check | 0.098 | 18:40:38 |
| Air Blank | 0.000 | 18:41:08 |
| Subject Test | 0.291* | 18:44:16 |
| Air Blank | 0.000 | 18:44:57 |
| Air Blank | 0.000 | 18:45:24 |
| Subject Test | XXX** | 18:45:39 |
| Air Blank | 0.000 | 18:46:12 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY Goodyear PD
DATE 11/30/21 TIME 18:46
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

11/30/2021

18:46:29

Standard Lot#: 27220100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 18:46:58 |
| 0.100 Cal Check | 0.097 | 18:47:19 |
| Air Blank | 0.000 | 18:47:49 |

Preventative Maintenance
Performed -- Timer Reset