

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY Goodyear PD
DATE 5/2/2022 TIME 1233
INTOXILYZER SERIAL # 80-001858 LOCATION 11N 145TH AVE

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
() 4. Air blank completed.
() 5. Calibration check completed. Test results 0. 101 AC.
() 6. Air blank completed.
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____ 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
05/02/2022 12:33:12

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:33:46
0.100 Cal Check	0.101	12:34:07
Air Blank	0.000	12:34:37

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K. Miyazaki 1139 AGENCY Coolidge PD
DATE 5/2/2022 TIME 12:36
INTOXILYZER SERIAL # 80-00/858 LOCATION 11 N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE [Signature] 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/02/2022

12:36:23

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:37:18
Diagnostic Test	Pass	12:37:53
Air Blank	0.000	12:38:21
0.100 Cal Check	0.100	12:38:41
Air Blank	0.000	12:39:11
Subject Test	0.000	12:39:26
Air Blank	ABT*	12:39:33
Air Blank	0.000	12:40:01

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/02/2022

12:40:04

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	12:40:33
Air Blank	0.000	12:41:01

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/02/2022

12:41:06

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	12:41:39
Air Blank	0.000	12:42:07

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
05/02/2022

12:42:11

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:42:59
Diagnostic Test	Pass	12:43:34
Air Blank	0.000	12:44:02
0.100 Cal Check	0.100	12:44:23
Air Blank	0.000	12:44:52
Subject Test	0.400*	12:48:00
Air Blank	0.000	12:48:46
Air Blank	0.000	12:49:13
Subject Test	XXX**	12:49:26
Air Blank	0.000	12:50:00

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K M WYATT 1139 AGENCY GOODYEAR PD
DATE 5/2/2022 TIME 12:50
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145 TH AVE

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K M WYATT 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
05/02/2022 12:50:15

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:50:45
0.100 Cal Check	0.098	12:51:06
Air Blank	0.000	12:51:36

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K MIYAZAWA 1139 AGENCY Goodyear PD

DATE 5/30/22 TIME _____

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO _____

COMMENTS

SIGNATURE

K Miyazawa 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
05/30/2022

19:56:36

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:57:31
Diagnostic Test	Pass	19:58:06
Air Blank	0.000	19:58:33
0.100 Cal Check	0.100	19:58:54
Air Blank	0.000	19:59:24
Subject Test	0.007*	20:02:31
Air Blank	0.000	20:03:05
Air Blank	0.000	20:03:32
Subject Test	XXX**	20:03:47
Air Blank	0.000	20:04:24

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/30/2022

19:54:54

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	19:56:04
Air Blank	0.000	19:56:32

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/30/2022

19:53:15

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:54:22
Air Blank	0.000	19:54:50

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/30/2022

19:49:10

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:50:30
Diagnostic Test	Pass	19:51:05
Air Blank	0.000	19:51:32
0.100 Cal Check	0.100	19:51:53
Air Blank	0.000	19:52:23
Subject Test	0.000	19:52:36
Air Blank	ABT*	19:52:42
Air Blank	0.000	19:53:11

*Sequence Aborted

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazaki 1139 AGENCY Goodyear PD

DATE 5/30/22 TIME 2007

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
05/30/2022 20:07:00

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	20:07:30
0.100 Cal Check	0.099	20:07:51
Air Blank	0.000	20:08:20

Preventative Maintenance
Performed -- Timer Reset