

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazawa AGENCY 1139 GARDNER PD  
DATE 6/11/22 TIME 0:329  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0.099 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazawa

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/11/2022 03:29:47

-----  
Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139  
-----

QAS: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Operator: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	03:30:23
0.100 Cal Check	0.099	03:30:43
Air Blank	0.000	03:31:13

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazaki 1139 AGENCY Goodyear PD  
DATE 6/30/22 TIME 2104  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVB

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 106 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE K Miyazaki 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/30/2022

21:04:28

---

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	21:04:56
Air Blank	0.000	21:05:24

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/30/2022

21:05:31

---

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	21:06:07
Air Blank	0.000	21:06:35

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/30/2022

21:06:38

---

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:07:24
Diagnostic Test	Pass	21:07:59
Air Blank	0.000	21:08:27
0.100 Cal Check	0.099	21:08:48
Air Blank	0.000	21:09:18
Subject Test	0.069*	21:12:25
Air Blank	0.000	21:12:58
Air Blank	0.000	21:13:25
Subject Test	XXX**	21:13:39
Air Blank	0.000	21:14:12

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/30/2022

21:00:52

---

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:01:42
Diagnostic Test	Pass	21:02:17
Air Blank	0.000	21:02:45
0.100 Cal Check	0.099	21:03:06
Air Blank	0.000	21:03:36
Subject Test	0.000	21:03:49
Air Blank	ABT*	21:03:56
Air Blank	0.000	21:04:24

\*Sequence Aborted

Not a Successfully Completed Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear  
DATE 6/30/22 TIME 2114  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazato 1139



INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/30/2022

21:14:35

-----  
Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139  
-----

QAS: K. MIYAZATO #1139

GOODYEAR PD  
-----

Operator: K. MIYAZATO #1139

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	21:15:06
0.100 Cal Check	0.098	21:15:27
Air Blank	0.000	21:15:57

Preventative Maintenance  
Performed -- Timer Reset