



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. I800018580011

Page 1 of 1

AGENCY Goodyear Police Department
 Goodyear, AZ 85338
OFFICER MIYAZATO, #1139
DATE June 28, 2022



INSTRUMENT

80001858

INSTRUMENT LOCATION

Goodyear Police Department

MAINTENANCE REQUESTED

Repair and Preventative Maintenance

MAINTENANCE PERFORMED

Parts Replaced:

Breath Hose

Preventative Maintenance Performed:

Inspection, Calibration, Verification

Erin E. Boone

ERIN E. BOONE, #5823, Forensic Scientist
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2281 eboone@azdps.gov

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear
DATE 6/30/22 TIME 2:28
INTOXILYZER SERIAL # 80-001858 LOCATION 11N 145TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Kuro 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/30/2022

21:28:11

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:29:04
Diagnostic Test	Pass	21:29:39
Air Blank	0.000	21:30:06
0.100 Cal Check	0.099	21:30:27
Air Blank	0.000	21:30:57
Subject Test	0.000	21:31:11
Air Blank	ABT*	21:31:18
Air Blank	0.000	21:31:46

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/30/2022

21:31:48

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:32:40
Diagnostic Test	Fail*	21:32:50
Air Blank	RFI**	21:32:57

* Analytical Stability Test

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
06/30/2022

21:33:58

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:34:51
Diagnostic Test	Pass	21:35:26
Air Blank	0.000	21:35:53
0.100 Cal Check	0.101	21:36:14
Air Blank	0.000	21:36:44
Subject Test	0.094*	21:39:51
Air Blank	0.000	21:40:24
Air Blank	0.000	21:40:51
Subject Test	0.019	21:41:06
Air Blank	IPS**	21:41:12
Air Blank	0.000	21:41:41

*Deficient Sample

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/30/2022

21:42:05

Standard Lot#: 15421100A4

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:43:23
Diagnostic Test	Pass	21:43:58
Air Blank	0.000	21:44:25
0.100 Cal Check	0.099	21:44:46
Air Blank	0.000	21:45:16
Subject Test	0.152*	21:48:23
Air Blank	0.000	21:48:56
Air Blank	0.000	21:49:23
Subject Test	XXX**	21:49:36
Air Blank	0.000	21:50:11

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear
DATE 6/30/22 TIME 2:50
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVB

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
() 4. Air blank completed.
() 5. Calibration check completed. Test results 0. 099 AC.
() 6. Air blank completed.
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazato 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
06/30/2022 21:50:32

Standard Lot#: 15421100A4
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:51:02
0.100 Cal Check	0.099	21:51:23
Air Blank	0.000	21:51:52

Preventative Maintenance
Performed -- Timer Reset