

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear PD  
DATE 9/23/22 TIME 0522  
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
() 4. Air blank completed.  
() 5. Calibration check completed. Test results 0.099 AC.  
() 6. Air blank completed.  
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazato 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/23/2022

05:22:06

-----  
Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

-----  
QAS: K. MIYAZATO #1139

GOODYEAR PD

-----  
Operator: K. MIYAZATO #1139

GOODYEAR PD

-----  
Test                    g/210L                    Time  
-----  
Air Blank                0.000                    05:22:36  
0.100 Cal Check        0.099                    05:22:56  
Air Blank                0.000                    05:23:26

-----  
Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY GOODYEAR  
DATE 9/30/22 TIME 0424  
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. ~~000~~ 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE Kmi 1139

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001658  
Core Version: 8105.48  
09/30/2022 04:24:44

-----  
Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139  
-----

QAS: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Operator: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Subject: SOAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200  
-----

15 Minute Deprivation Period? Yes  
-----

Test	g/210L	Time
Air Blank	0.000	04:25:47
Diagnostic Test	Pass	04:26:24
Air Blank	0.000	04:26:52
0.100 Cal Check	0.098	04:27:13
Air Blank	0.000	04:27:42
Subject Test	0.000	04:28:01
Air Blank	ABT*	04:28:08
Air Blank	0.000	04:28:37

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/30/2022

04:29:08

---

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139

GOODYEAR PD

---

Operator: K. MIYAZATO #1139

GOODYEAR PD

---

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	04:30:02
Air Blank	0.000	04:30:31

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
09/30/2022

04:30:36

---

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	04:31:26
Air Blank	RFI*	04:31:47

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
09/30/2022

04:32:14

---

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

---

QAS: K. MIYAZATO #1139  
GOODYEAR PD

---

Operator: K. MIYAZATO #1139  
GOODYEAR PD

---

Subject: SQAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	04:33:23
Diagnostic Test	Pass	04:33:58
Air Blank	0.000	04:34:26
0.100 Cal Check	0.099	04:34:47
Air Blank	0.000	04:35:16
Subject Test	0.329*	04:38:24
Air Blank	0.000	04:38:58
Air Blank	0.000	04:39:25
Subject Test	XXX**	04:39:38
Air Blank	0.000	04:40:12

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY GOODYEAR  
DATE 9/30/22 TIME 0440  
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 097 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE KM 1139



INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
09/30/2022 04:40:36

-----  
Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139  
-----

QAS: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Operator: K. MIYAZATO #1139  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	04:41:06
0.100 Cal Check	0.097	04:41:27
Air Blank	0.000	04:41:56

Preventative Maintenance  
Performed -- Timer Reset