

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY Gandy SAR PD
DATE 12/1/2022 TIME 0441
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S EMERALD PKWAY

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

K Miyazawa 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2022

04:41:23

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:42:27
Diagnostic Test	Pass	04:43:02
Air Blank	0.000	04:43:30
0.100 Cal Check	0.100	04:43:51
Air Blank	0.000	04:44:20
Subject Test	0.000	04:44:33
Air Blank	ABT*	04:44:40
Air Blank	0.000	04:45:08

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2022

04:45:10

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	04:45:59
Air Blank	0.000	04:46:27

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2022

04:46:31

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	04:47:14
Air Blank	0.000	04:47:42

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2022

04:47:44

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:48:41
Diagnostic Test	Pass	04:49:16
Air Blank	0.000	04:49:43
0.100 Cal Check	0.099	04:50:04
Air Blank	0.000	04:50:33
Subject Test	0.198*	04:53:41
Air Blank	0.000	04:54:17
Air Blank	0.000	04:54:44
Subject Test	XXX**	04:54:57
Air Blank	0.000	04:55:31

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato¹¹³⁹ AGENCY Goodyear PD
DATE 12/1/2022 TIME 0455
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

- (1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(4. Air blank completed.
(5. Calibration check completed. Test results 0.098 AC.
(6. Air blank completed.
(7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
12/01/2022 04:55:49

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	04:56:19
0.100 Cal Check	0.098	04:56:40
Air Blank	0.000	04:57:10

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazato AGENCY Goodyear PD
DATE 12/29/22 TIME 2341
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

K Miyazato 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/29/2022

23:41:07

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:42:35
Diagnostic Test	Pass	23:43:10
Air Blank	0.000	23:43:37
0.100 Cal Check	0.099	23:43:58
Air Blank	0.000	23:44:28
Subject Test	0.000	23:44:55
Air Blank	ABT*	23:45:04
Air Blank	0.000	23:45:32

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
12/29/2022 23:45:39

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	23:46:19
Air Blank	0.000	23:46:47

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/29/2022

23:46:51

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	23:47:28
Air Blank	0.000	23:47:56

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/29/2022

23:47:59

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:48:59
Diagnostic Test	Pass	23:49:34
Air Blank	0.000	23:50:01
0.100 Cal Check	0.099	23:50:22
Air Blank	0.000	23:50:52
Subject Test	0.105*	23:53:59
Air Blank	0.000	23:54:36
Air Blank	0.000	23:55:04
Subject Test	XXX**	23:55:15
Air Blank	0.000	23:55:49

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato AGENCY Goodyear PD
DATE 12/29/22 TIME 2356
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S Estrella Pkwy

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution-100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kurt 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
12/29/2022 23:56:43

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	23:57:14
0.100 Cal Check	0.098	23:57:35
Air Blank	0.000	23:58:04

Preventative Maintenance
Performed -- Timer Reset