

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KMiyazawa 1139 AGENCY Gouyean
DATE 1/31/2023 TIME 2141
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE KMiyazawa 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
01/31/2023 21:41:06

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:42:18
0.100 Cal Check	0.100	21:42:39
Air Blank	0.000	21:43:09

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K M MAZADU 1139 AGENCY GOODYEAR
DATE 11/31/2023 TIME 2145
INTOXILYZER SERIAL # 80-000312 LOCATION 112145 TH AVE

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

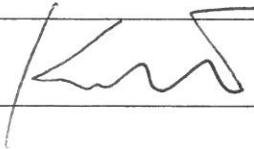
- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE  1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

01/31/2023

21:45:30

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:46:20
Diagnostic Test	Pass	21:46:55
Air Blank	0.000	21:47:23
0.100 Cal Check	0.100	21:47:44
Air Blank	0.000	21:48:14
Subject Test	0.000	21:48:30
Air Blank	ABT*	21:48:36
Air Blank	0.000	21:49:05

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

01/31/2023

21:49:06

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	21:49:36
Air Blank	0.000	21:50:04

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

01/31/2023

21:50:08

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	21:50:42
Air Blank	RFI*	21:51:01

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

01/31/2023

21:51:34

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:52:23
Diagnostic Test	Pass	21:52:58
Air Blank	0.000	21:53:25
0.100 Cal Check	0.100	21:53:46
Air Blank	0.000	21:54:16
Subject Test	0.329*	21:57:23
Air Blank	0.000	21:57:56
Air Blank	0.000	21:58:23
Subject Test	XXX**	21:58:35
Air Blank	0.000	21:59:09

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazawa AGENCY Goodyear
DATE 1/31/2023 TIME 2159
INTOXILYZER SERIAL # 80-000312 LOCATION 11N 145TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kand 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

01/31/2023

21:59:22

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:59:52
0.100 Cal Check	0.099	22:00:13
Air Blank	0.000	22:00:43

Preventative Maintenance
Performed -- Timer Reset