

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazaki AGENCY Goodyear
DATE 8/2/23 TIME 0358
INTOXILYZER SERIAL # 80-000312 LOCATION 11N 145 TH AVE

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
() 4. Air blank completed.
() 5. Calibration check completed. Test results 0. 097 AC.
() 6. Air blank completed.
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/02/2023

03:58:58

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	03:59:28
0.100 Cal Check	0.097	03:59:49
Air Blank	0.000	04:00:19

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K MIYAZATO AGENCY GOODYEAR
DATE 8/2/23 TIME 0342
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE K Miyazato

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
08/02/2023

03:42:43

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	03:43:50
Diagnostic Test	Pass	03:44:25
Air Blank	0.000	03:44:53
0.100 Cal Check	0.099	03:45:14
Air Blank	0.000	03:45:44
Subject Test	0.000	03:45:59
Air Blank	ABT*	03:46:06
Air Blank	0.000	03:46:34

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
08/02/2023

03:46:39

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	03:47:25
Air Blank	0.000	03:47:53

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
08/02/2023

03:47:57

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP, TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	03:48:56
Air Blank	0.000	03:49:25

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/02/2023

03:50:12

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	03:51:29
Diagnostic Test	Pass	03:52:03
Air Blank	0.000	03:52:31
0.100 Cal Check	0.098	03:52:52
Air Blank	0.000	03:53:22
Subject Test	0.253*	03:56:29
Air Blank	0.000	03:57:08
Air Blank	0.000	03:57:36
Subject Test	XXX**	03:57:57
Air Blank	0.000	03:58:31

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
08/02/2023

03:39:22

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	03:40:08
0.100 Cal Check	0.098	03:40:29
Air Blank	0.000	03:40:59

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear P.D.
DATE 8/31/2023 TIME 0500
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145 TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE [Signature] 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:00:57

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO

#1139

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	05:01:46
Diagnostic Test	Pass	05:02:21
Air Blank	0.000	05:02:49
0.100 Cal Check	0.099	05:03:10
Air Blank	0.000	05:03:40
Subject Test	0.000	05:03:56
Air Blank	ABT*	05:04:05
Air Blank	0.000	05:04:34

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:04:36

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	05:05:26
Diagnostic Test	IPS*	05:05:46
Air Blank	0.000	05:06:14

* Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:06:26

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	05:07:06
Air Blank	0.000	05:07:34

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:17:03

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	05:17:50
Diagnostic Test	Pass	05:18:25
Air Blank	0.000	05:18:52
0.100 Cal Check	0.098	05:19:13
Air Blank	0.000	05:19:43
Subject Test	0.132*	05:22:51
Air Blank	0.000	05:23:23
Air Blank	0.000	05:23:50
Subject Test	RGE**	05:24:03
Air Blank	0.000	05:24:48

*Deficient Sample

**Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:25:11

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	05:25:57
Diagnostic Test	Pass	05:26:33
Air Blank	0.000	05:27:00
0.100 Cal Check	0.098	05:27:21
Air Blank	0.000	05:27:51
Subject Test	0.530*	05:30:58
Air Blank	0.000	05:31:37
Air Blank	0.000	05:32:04
Subject Test	XXX**	05:32:18
Air Blank	0.000	05:32:52

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY Goodyear PD
DATE 8/31/2023 TIME 0533
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

() 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

() 4. Air blank completed.

() 5. Calibration check completed. Test results 0.096 AC.

() 6. Air blank completed.

() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K Miyazawa 1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

08/31/2023

05:33:05

Standard Lot#: 28021100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 05:33:35
0.100 Cal Check 0.096 05:33:56
Air Blank 0.000 05:34:26

Preventative Maintenance
Performed -- Timer Reset