

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K. MIYAZATO AGENCY GOODYEAR  
DATE 8/3/2023 TIME 2047  
INTOXILYZER SERIAL # 80-001858 LOCATION 11N 145TH AVB

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE [Signature] 1139

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Kory Miyazaki AGENCY Goodyear  
DATE 8/3/23 TIME 2102  
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145 TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
() 4. Air blank completed.  
() 5. Calibration check completed. Test results 0. 100 AC.  
() 6. Air blank completed.  
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1139

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023

20:47:29

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Subject: SOAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:48:19
Diagnostic Test	Pass	20:48:57
Air Blank	0.000	20:49:24
0.100 Cal Check	0.100	20:49:45
Air Blank	0.000	20:50:14
Subject Test	0.000	20:50:31
Air Blank	ABT*	20:50:38
Air Blank	0.000	20:51:06

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023

20:51:13

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Subject: SOAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	20:51:49
Air Blank	0.000	20:52:18

\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023

20:52:25

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Subject: SOAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	20:53:11
Air Blank	0.000	20:53:39

\*RFI Detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023 20:53:51

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Subject: SOAP,TEST  
DOB: 04/02/1973  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:54:38
Diagnostic Test	Pass	20:55:16
Air Blank	0.000	20:55:43
0.100 Cal Check	0.100	20:56:04
Air Blank	0.000	20:56:33
Subject Test	0.067*	20:59:41
Air Blank	0.000	21:00:16
Air Blank	0.000	21:00:43
Subject Test	XXX**	21:00:58
Air Blank	0.000	21:01:32

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023 20:44:13

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	20:44:44
0.100 Cal Check	0.099	20:45:05
Air Blank	0.000	20:45:35

Preventative Maintenance  
Performed -- Timer Reset

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
08/03/2023 21:02:03

Standard Lot#: 28021100A1  
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139  
GOODYEAR PD

Operator: K. MIYAZATO #1139  
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:02:33
0.100 Cal Check	0.100	21:02:54
Air Blank	0.000	21:03:23

Preventative Maintenance  
Performed -- Timer Reset