

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K Miyazawa 1139 AGENCY Goodyear
DATE 10/18/23 TIME 0601
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. ~~100~~ AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE K Miyazawa 1139

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K Miyazato 1139 AGENCY Goodyear
DATE 10/18/23 TIME 0616
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
() 4. Air blank completed.
() 5. Calibration check completed. Test results 0.098 AC.
() 6. Air blank completed.
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kent 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023 06:08:53

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

OAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SOAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	06:09:44
Diagnostic Test	Pass	06:10:22
Air Blank	0.000	06:10:49
0.100 Cal Check	0.098	06:11:10
Air Blank	0.000	06:11:40
Subject Test	0.071*	06:14:47
Air Blank	0.000	06:15:20
Air Blank	0.000	06:15:47
Subject Test	XXX**	06:16:02
Air Blank	0.000	06:16:35

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully
Completed
Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023 05:57:13

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

OAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	05:57:46
0.100 Cal Check	0.098	05:58:07
Air Blank	0.000	05:58:36

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023 06:06:19

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

OAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SOAP,TEST
DOB: 04/02/1973
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	1PS*	06:06:53
Air Blank	0.000	06:07:21

*Improper Sample

Not a Successfully
Completed
Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023

06:07:29

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

DAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SOAP, TEST
DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RF1*	06:08:16
Air Blank	0.000	06:08:45

*RF1 Detect

Not a Successfully
Completed
Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023

06:01:47

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

DAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SOAP, TEST
DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	06:02:48
Diagnostic Test	Pass	06:03:26
Air Blank	0.000	06:03:53
0.100 Cal Check	0.098	06:04:14
Air Blank	0.000	06:04:43
Subject Test	0.000	06:05:12
Air Blank	ABT*	06:05:22
Air Blank	0.000	06:05:50

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
10/18/2023

06:16:59

Standard Lot#: 28021100A1
Last Changed By: K. MIYAZATO #1139

DAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	06:17:29
0.100 Cal Check	0.098	06:17:49
Air Blank	0.000	06:18:19

Preventative Maintenance
Performed -- Timer Reset