

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST KM142220 1139 AGENCY GONDYCAR
DATE 8 (30/24) TIME 1836
INTOXILYZER SERIAL# 80-00/858 LOCATION // 145 TH ANS
(1) 1. Display Reads "PUSH BUTTON TO START"
DIAGNOSTIC TESTS (') 1. Clock time check. () 2. Date check.
OPERATIONAL TESTS () 1. Alcohol-free subject test result 0
COMMENTS
SIGNATURE
DPS Form Exh G-4 (Rev 05-01)

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

08/30/2024 18:36:51

Standard Lot#: 24923100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:37:48
Diagnostic Test	Pass	18:38:23
Air Blank	0.000	18:38:51
0.100 Cal Check	0.100	18:39:12
Air Blank	0.000	18:39:41
Subject Test	0.000	18:39:58
Air Blank	ABT*	18:40:06
Air Blank	0.000	18:40:34

^{*}Sequence Aborted

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

08/30/2024 18:40:38

Standard Lot#: 24923100A1
Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	18:41:17
Air Blank	0.000	18:41:45

^{*}Improper Sample

.

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

08/30/2024

18:41:48

S	tan	dard	Lo	七井	24	92	3	1	0	() A	.1	
0.000				-				_		_			_

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP, TEST

DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

 Test
 g/210L
 Time

 Air Blank
 RFI*
 18:42:23

 Air Blank
 0.000
 18:42:51

^{*}RFI Detect

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

08/30/2024 18:42:56

Standard Lot#: 24923100A1 Last Changed By: K. MIYAZATO	#1139
QAS: K. MIYAZATO GOODYEAR PD	#1139
Operator: K. MIYAZATO GOODYEAR PD	#1139

Subject: SQAP, TEST DOB: 04/02/1973

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

7

^{*}Deficient Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

^{**}Invalid Sample

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K MIGAZAT - 1139 AGENCY GODDY ZAR
QA SPECIALIST / 191927 / AGENCY C 2019 2010
DATE $\sqrt{30/29}$ TIME $\sqrt{850}$
INTOXILYZER SERIAL# 86-80 858 LOCATION // N 145 TH AND
(1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution — (o OR
Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
 Intoxilyzer 8000 display reads "PUSH BUTTON TO START" Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed. 5. Calibration check completed. Test results 0. AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.
SIGNATURE (139)
DPS Form Exh G-2 (Rev 05-01)

Location: 11 N 145TH AVE Serial Number: 80-001858 Core Version: 8105.48

08/30/2024 18:50:57

Standard Lot#: 24923100A1

Last Changed By: K. MIYAZATO #1139

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:51:26
0.100 Cal Check	0.100	18:51:47
Air Blank	0.000	18:52:16

Preventative Maintenance Performed -- Timer Reset



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790 217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID:

15958

Part #:

BAC105L100T

Cylinder Size:

105L

Lot Number:

24923100A1

Expiration:

10/5/2025

0.100 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Reported

Accuracy

Analytical

Component:

Concentration:

(U, k=2):

Method:

Ethanol Nitrogen

260 ppm Balance

+/-0.002 BAC(G/210L) NDIR [5.2 ppm]

CMI Inc.

Distributed by:

316 East Ninth Street Owensboro, KY 42303 Phone 866-835-0690 www.alcoholtest.com

*Traceable to:

Certified Reference Material - 261.0 µmol/mol

Ethanol in Nitrogen - Serial No. ND7017 Lot No. 080722E2

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Accreditation #61895

The calibration results within this certificate were obtained at the facility listed above using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service. This certificate applies only to the items described and shall not be reproduced other than in full, without written approval from the issuing facility.