

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Michael Guilbault AGENCY GOODYEAR PD  
DATE 6/18/2024 TIME 1549  
INTOXILYZER SERIAL # 80-001880 LOCATION 11 N 145<sup>TH</sup> AVE

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.100 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 9000  
Location: 11 N 145TH AVENUE  
Serial Number: 80-001880  
Core Version: 8105.48  
06/18/2024

15:49:44 *JK*

-----  
Standard Lot#: 24923100A1  
Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	15:50:13
0.100 Cal Check	0.100	15:50:34
Air Blank	0.000	15:51:04

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Michael Guilbault AGENCY GOODYEAR PD  
DATE 6/18/2024 TIME 1552  
INTOXILYZER SERIAL # 80-001880 LOCATION 11 N 145TH AVE

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

---

---

---

---

SIGNATURE Michael Guilbault

INTOXILYZER 8000  
Location: 11 N 145TH AVENUE  
Serial Number: 80-001880  
Core Version: 8105.48  
06/18/2024

15:52:43

---

Standard Lot#: 24923100A1  
Last Changed By: M. GUILBAULT #1144

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:54:01
Diagnostic Test	Pass	15:54:37
Air Blank	0.000	15:55:04
0.100 Cal Check	0.100	15:55:25
Air Blank	0.000	15:55:54
Subject Test	0.000	15:56:13
Air Blank	IPS*	15:56:19
Air Blank	RFI**	15:56:28

\*Improper Sample  
\*\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVENUE  
Serial Number: 80-001880  
Core Version: 8105.48  
06/18/2024

15:56:51

---

Standard Lot#: 24923100A1  
Last Changed By: M. GUILBAULT #1144

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:58:08
Diagnostic Test	Pass	15:58:43
Air Blank	0.000	15:59:10
0.100 Cal Check	0.100	15:59:31
Air Blank	0.000	16:00:01
Subject Test	0.175*	16:03:08
Air Blank	0.000	16:03:42
Air Blank	0.000	16:04:09
Subject Test	XXX**	16:04:29
Air Blank	0.000	16:05:04

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVENUE  
Serial Number: 80-001880  
Core Version: 8105.48  
06/18/2024 16:05:10

---

Standard Lot#: 24923100A1  
Last Changed By: M. GUILBAULT #1144

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: SQAP, TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:06:05
Diagnostic Test	Pass	16:06:40
Air Blank	ABT*	16:07:02
Air Blank	0.000	16:07:30

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 5000  
Location: 11 N 145TH AVENUE  
Serial Number: 80-001880  
Core Version: 8105.48  
05/18/2024

16:07:44

-----  
Standard Lot#: 24923100A1  
Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:08:14
0.100 Cal Check	0.099	16:08:34
Air Blank	0.000	16:09:05

Preventative Maintenance  
Performed -- Timer Reset